

INSTRUCTIONS FOR USE
MOUNJARO® KwikPen® [muhn-JAHR-oh]
tirzepatide
solution for subcutaneous injection

multi-dose single-patient-use prefilled pen

Each pen contains 4 fixed doses, one dose taken weekly.

2.5 mg/0.6 mL prefilled pen

5 mg/0.6 mL prefilled pen

7.5 mg/0.6 mL prefilled pen

10 mg/0.6 mL prefilled pen

12.5 mg/0.6 mL prefilled pen

15 mg/0.6 mL prefilled pen

This Instructions for Use contains information on how to inject MOUNJARO.



Lilly

Important information you need to know before injecting MOUNJARO

Read the Instructions for Use and Consumer Medicine Information (CMI) carefully before you start injecting MOUNJARO KwikPen (Pen) and each time you get another new Pen. There may be new information. This information does not take the place of talking to your healthcare provider about your medical condition or your treatment.

MOUNJARO Pen is a disposable multi-dose single-patient-use prefilled pen. **The Pen contains 4 fixed doses, one dose taken weekly.** Inject 0.6 mL in a single weekly injection, under the skin (subcutaneously).


After 4 doses, throw away (discard) the Pen, including the unused medication. The Pen will prevent you from dialing a full 0.6 mL dose after you have given yourself 4 weekly doses. **Do not** inject the leftover medication. **Do not** transfer MOUNJARO from your Pen into a syringe.

Do not share your MOUNJARO Pen with other people, even if the pen needle has been changed. You may give other people a serious infection or get a serious infection from them.

People who are blind or have vision problems should not use the Pen without help from a person trained to use the Pen.

Guide to parts

Parts of the MOUNJARO Pen

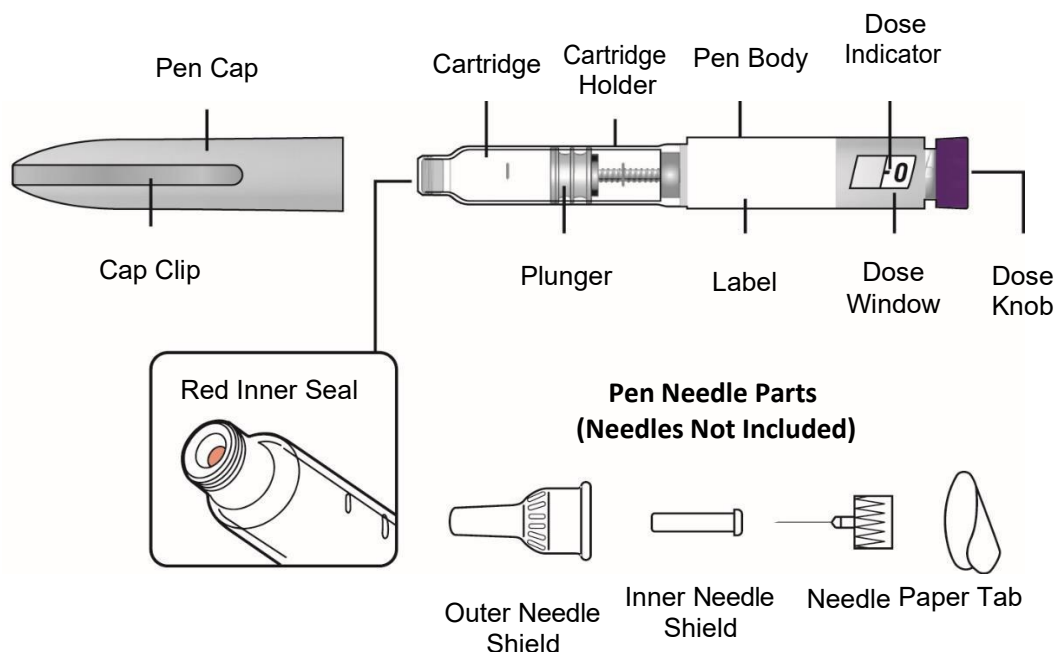


KwikPen compatible needle (If you do not know what pen needle to use, talk to your healthcare professional)

Supplies needed to give your injection


- MOUNJARO Pen
- KwikPen compatible needle
- Gauze or cotton ball
- sharps disposal container or household container

Preparing to inject MOUNJARO

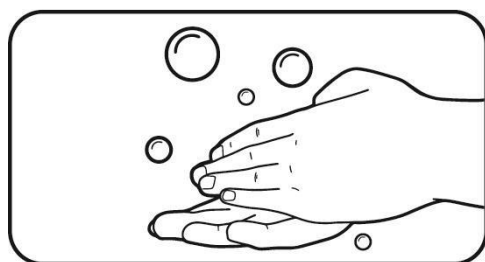


Step 1:

- Wash your hands with soap and water.

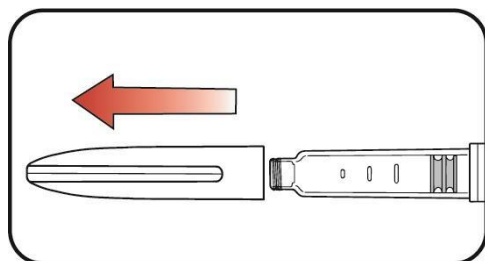


Step 2:

- Pull the Pen cap straight off.
- Inspect the Pen and label. **Do not** use if:
 - the medication name or dose strength does not match your prescription.
 - the Pen is expired (EXP) or looks damaged.
 - the medication has been frozen, has particles, is cloudy, or is discoloured. MOUNJARO should be colourless to slightly yellow.

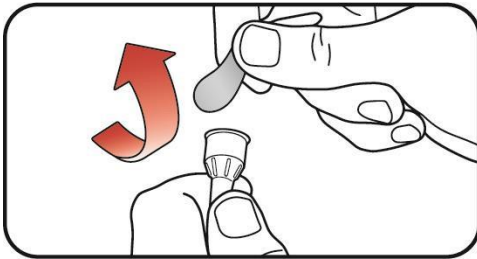


Step 3:

- **Select a new pen needle.** Always use a new pen needle for each injection to help prevent infections and blocked needles.
- Pull off the paper tab from the outer needle shield.

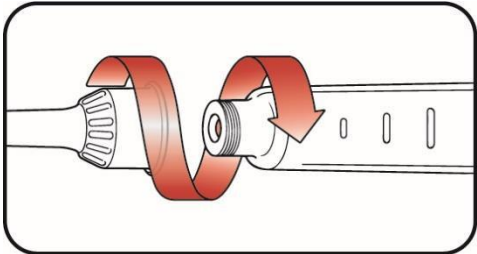

Step 4:

- Push the capped pen needle straight onto the Pen and twist the pen needle on until it is tight.

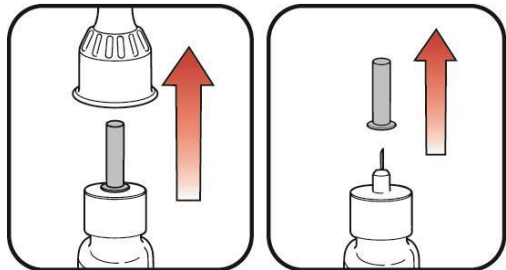

a. Outer needle shield

Inner needle shield

Step 5:


- Pull off the outer needle shield and keep it. This will be reused.
- Pull off the inner needle shield. Put it in your household trash.

CHECKING THE FLOW/PRIMING (do this before each injection)

Step 6:


- Slowly turn the dose knob until you hear **2 clicks** and the  extended line is shown in the dose window.


This is the prime position. It can be corrected by turning the dose knob in either direction until the prime position lines up to the dose indicator.

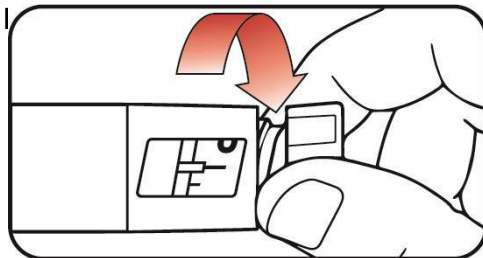

Step 7:

- Hold your Pen with the needle pointing up.
- Tap the cartridge holder gently to collect air bubbles at the top.

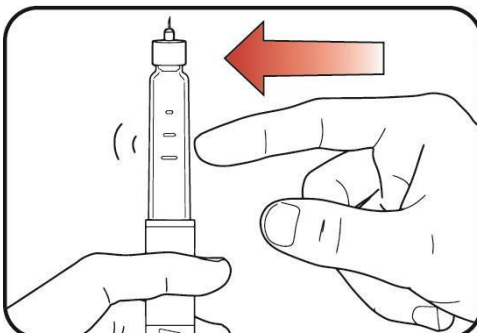

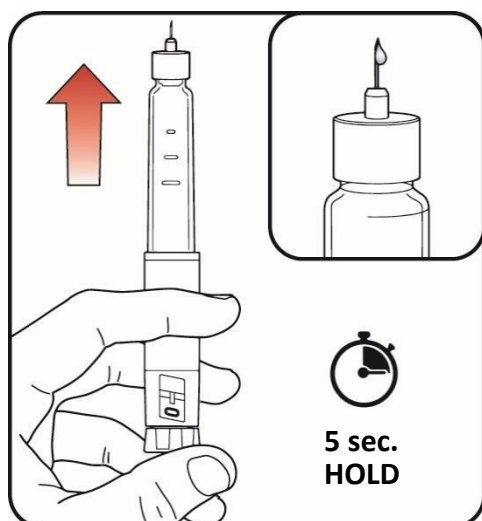


Step 8:

- Release some medicine into the air by **pushing the dose knob in** until it stops, then **slowly count to 5 while holding the dose knob**. The **0** icon must be shown in the dose window. **Do not** inject into your body.

Priming removes air from the cartridge and makes sure that your Pen is working correctly. Your Pen has been primed if a small amount of medicine comes out of the tip of the pen needle.


- If you do not see medicine, repeat **steps 6-8**, no more than 2 additional times.
- If you still do not see medicine, then change the pen needle and repeat **steps 6-8**, no more than 1 additional time.
- If you still do not see medicine, contact Lilly at 1-800-4 LILLY (1-800-454-559) in Australia, or 0800 500 056 in New Zealand.

Injecting MOUNJARO

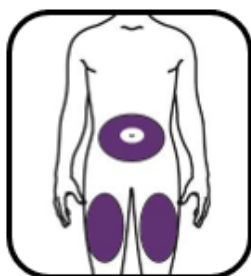




Step 9:

- Choose an injection site.
 - You or another person can inject the medicine in your thigh or stomach (abdomen) at least 5 centimeters from the belly button.
- Change (rotate)** your injection site each week. You may use the same area of your body but be sure to choose a different injection site in that area.

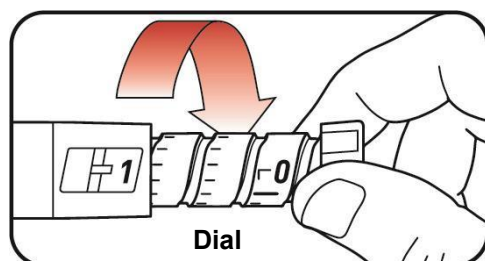

Step 10:

- Turn the dose knob until it stops and the **1** icon is shown in the dose window. The **1** icon is equal to a full 0.6 mL dose.
- Note:** Failure to correctly turn the dose knob fully to the **1** icon will result in injection of a lower dose of MOUNJARO. Ensure the **1** icon is shown in the dose window prior to injecting MOUNJARO to avoid underdosing (injecting too little medicine).

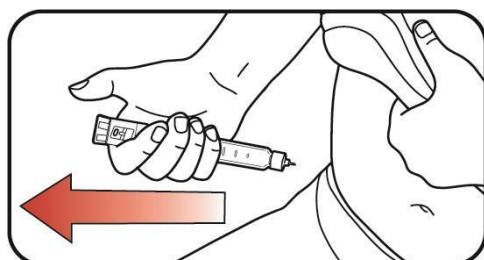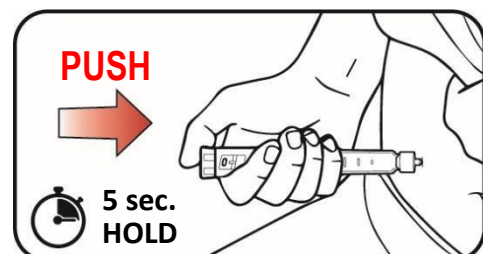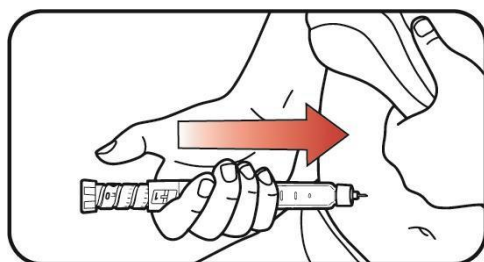

Step 11:

- a. Insert the needle into your skin.
- b. Inject the medicine by **pushing the dose knob in** until it stops then **slowly count to 5 while holding the dose knob**. The  icon must be shown in the dose window before removing the needle.



Step 12:

- Pull the needle out of your skin. A drop of medicine on the needle tip is normal. It will not affect your dose.
- Confirm the  icon is in the dose window.
If you see the  icon in the dose window, you have received the full dose.

If you do not see the  icon in the dose window, insert the needle back into your skin and finish your injection. **Do not** redial the dose.

If you still do not think you received the full dose, **do not** start over or repeat the injection. Contact Lilly at 1-800-4 LILLY (1-800-454-559) in Australia, or 0800 500 056 in New Zealand.

After your MOUNJARO injection

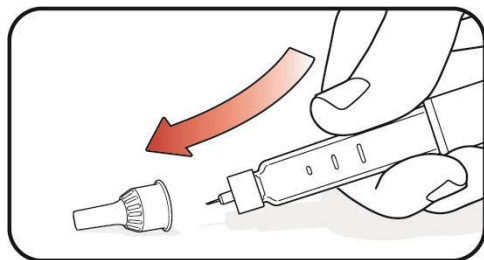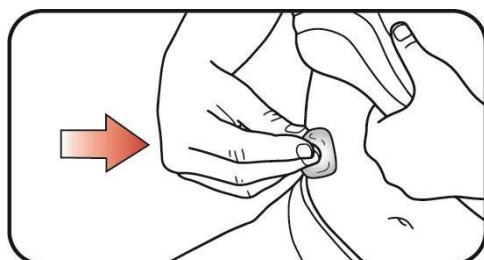


Step 13:

- If you see blood after you pull the needle out of your skin, lightly press the injection site with gauze or a cotton ball.
Do not rub the injection site.


Step 14:

- Carefully replace the outer needle shield.

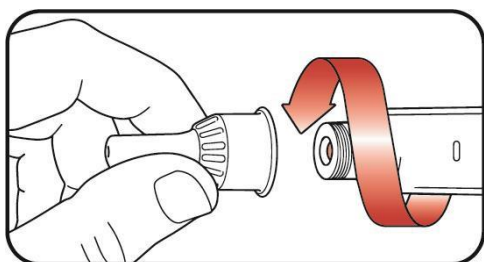**Step 15:**

- Unscrew the capped needle and put the needle in a sharps container (see "Disposing of MOUNJARO and needles" section).

Do not store the Pen with the needle attached to prevent leaking, blocking the needle, and air from entering the Pen.

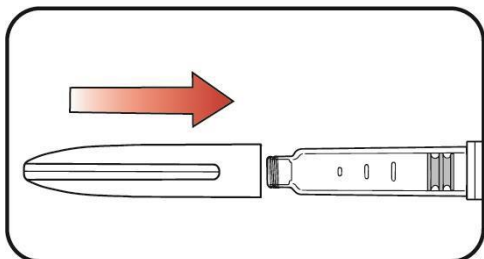**Step 16:**

- Replace the Pen cap.

Do not store the Pen without the pen cap attached.

Storing your MOUNJARO Pen

Unused Pens:

- Store **unused Pens** in the **refrigerator** between 2°C to 8°C.
- Unused Pens may be used until the expiration date printed on the label if the Pen has been kept in the refrigerator.
- **Do not** freeze your Pen. Throw away (discard) the Pen if it has been frozen.

Used Pens:

- You may store your **used Pen** at **room temperature** up to 30°C after your injection.
- Keep away from heat and light.
- Keep your Pen and needles out of the sight and reach of children.
- Throw away the Pen 30 days after first use even though the Pen has medication left in it.
- Throw away the Pen after receiving 4 weekly doses. Attempting to inject any leftover medicine could result in an incomplete dose even though the Pen still has medication left in it.

Disposing of MOUNJARO and pen needles

- Put your used pen needles in a sharps disposal container right away after use.
- **Do not** throw away (discard) loose pen needles in your household trash.
- Discard the used Pen in your household trash or sharps disposal container after you have removed the needle.
- If you do not have a sharps disposal container, you may use a household container that is:
 - made of a heavy-duty plastic,
 - can be closed with a tight-fitting, puncture-resistant lid, without sharps being able to come out,
 - upright and stable during use,
 - leak-resistant, and
 - properly labelled to warn of hazardous waste inside the container.
- When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container. There may be state or local laws about how you should throw away used needles and syringes. For more information about safe sharps disposal, and for specific information about sharps disposal in the state you live in, go to *RACGP - Safe sharps handling practices* (<https://www.racgp.org.au/running-a-practice/practice-standards/racgp-infection-prevention-and-control-guidelines/7-sharps/safe-sharps-handling-practices>).

- Do not dispose of your used sharps disposal container in your household trash unless your community guidelines permit this. Do not recycle your used sharps disposal container.

Troubleshooting

- If you cannot remove the pen cap, gently twist the pen cap back and forth, and then pull the pen cap straight off.
- If the dose knob is hard to push:
 - pushing the dose knob more slowly will make it easier to inject.
 - your needle may be blocked. Put on a new pen needle and prime the Pen.
 - you may have dust, food, or liquid inside the Pen. Throw the Pen away and get a new Pen.

Additional Information Australia:

If you have any questions or problems with MOUNJARO KwikPen:

- Call your healthcare provider
- Call 1-800-4 LILLY (1-800-454-559)
- Visit www.MOUNJARO.lilly.com.au



Scan this code to launch
www.MOUNJARO.lilly.com.au

Additional Information New Zealand:

If you have any questions or problems with MOUNJARO KwikPen:

- Call your healthcare provider
- Call 0800 500 056 in New Zealand
- Visit www.lilly.com/nz/

MOUNJARO® and KwikPen® are registered trademarks of Eli Lilly and Company.

Manufactured for:
Eli Lilly Australia Pty Ltd
Sydney, NSW 2000
Australia

Eli Lilly and Company (NZ) Limited
PO Box 109197 Newmarket
Auckland 1149
New Zealand

Copyright© 2025, Eli Lilly and Company. All rights reserved.

MOUNJARO KwikPen meets the current dose accuracy and functional requirements of ISO 11608-1.

Revised: August 2025

Medication Calendar

Use MOUNJARO 1 time a week.

I inject my weekly 0.6 mL dose on the dates below.

Write the day of the week you choose to inject.

Inject on this day each week (Example: Monday).

(Month/Day) (Month/Day) (Month/Day) (Month/Day)

--

--	--	--	--